

Assembling your mask



1. Place one filter into the guard making sure the blue side is facing outwards.



2. Mount the filter guard onto the mask.



3. Thread and adjust the headband.



4. breathe happy.


Removing the filter



1. Remove the filter assembly from the silicone mask body



2. Unclip the two halves of the filter holder

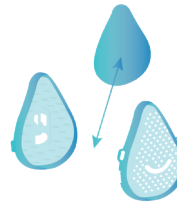

3. Remove the old filter from the assembly and dispose of it carefully*



4. You **MUST** wash your hands and sterilize you mask before inserting a new filter.

* Place the used filter in a plastic bag and secure the bag tightly. Place the bag into the rubbish bin. **ALWAYS** wash your hands immediately after disposing your filter. **ALWAYS** sterilize your mask, headband and filter holder after disposing your used filter and before inserting a new filter.

Sterilizing your mask



Please make sure you sterilize your breathe happy mask before initial use. Simply place the mask, headband and filter holder (not a filter) into a pan of boiling water for a minimum of five minutes. If your mask floats to the top of the water place a saucer or similar on top of it.

Once sterilized, remove carefully and allow to cool and air dry. Always sterilize your mask before inserting a new filter.