


## STEP 1

### Preparation

- Clear and clean a flat surface
- Get kit ready
- Timer at hand
- Tissues – blow nose
- Wash hands

## STEP 2

Unpack the kit

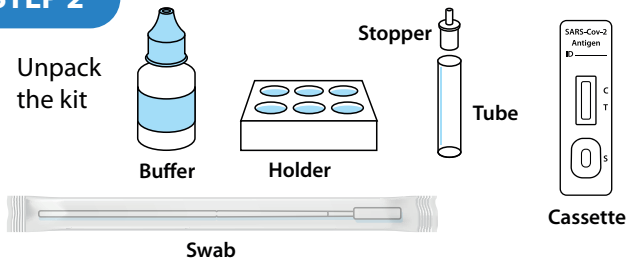


## STEP 3

Add 8 drops of buffer into the tube



## STEP 4


Tilt head, insert swab into nostril along floor of nose in backward direction. When you have reached nasopharynx, you will feel resistance. Rotate 4-5 times and remove.

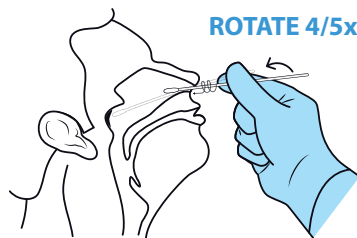

Nasopharyngeal


## STEP 5

Place swab into buffer tube, rotate swab several times to absorb buffer. Leave to stand for 1 minute.

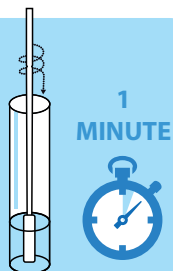


## STEP 6

Squeeze tube whilst removing swab

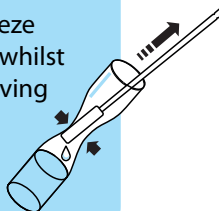

## STEP 7


Remove swab by squeezing TUBE firmly as swab is withdrawn. Place cap firmly on to the tube. Avoid touching top.

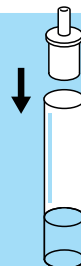

## STEP 8


Squeeze 3 or 4 drops from the tube into the bottom well marked S

Start timer: 15 minutes.

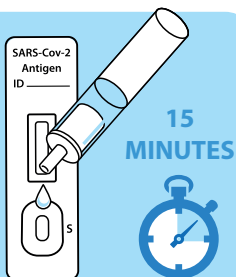

## STEP 9

When timer alarm sounds, read result.

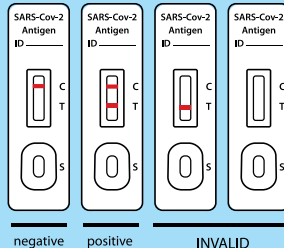

## STEP 10

Record/report as per local requirements.